



NOTES

1. Use 18 gauge wire for optional LI382 Corridor Zone Light wiring.
2. Main supply circuit: connect to 117 VAC, 60 Hz (40 watts max). Use 18 gauge wire minimum. SS106 Transformer rating is 24 VAC, 30 VA minimum.
3. Terminal identifications shown are not in actual order.
4. All common wires 18 gauge. All other wires 22 gauge.
5. See installation manual IL381 for complete system wiring information (this sheet is intended as a quick wiring guide only)



42 Central Drive, Farmingdale NY 11735-1202
 (631) 777-5500 Fax: (631) 777-5599
TOLL-FREE Technical Line 1-800-666-4800
 Email: info@alpha-comm.com

Typical Wiring Diagram for NC110N Series Audible-Visual (no intercom) Nurse-Call System with PK151A (or PK151N) Control Unit and a Single Master Annunciator Station. No Battery Back-Up.